



Vet1 VM10 Rapid Intervention Capnograph

Overview

The VM-10 veterinary end-respiratory capnograph is designed to provide continuous monitoring of the end-tidal CO2 concentration (EtCO2), capnography, and respiration rate (RR) of animals.

Power supply: 2XAAA battery/lithium battery; Small but not simple, plug and play; Intelligent data transfer.



Power On/Off Button	Airway Adapter
Turn off CapnoCOP by pressing the button "⏻" for 3s;	Large animals
Press the "⏻" button to mute the alarm;	Small animals
Configuration	Parameter Display
Press the "↑" "↓" button to configure CapnoCOP.	EtCO2
	AwRR
	Capnogram
Zero calibration	Status Display
Press and hold "⏻" "↓" together to trigger the Zero Calibration process.	Bluetooth
	Battery status

- High standard Accuracy CO2 measurement by dual wave length
- Visual and audible (buzzer) alarm
- Splash proof and rugged design for use in harsh conditions
- Real-time Capnogram display
- 4 buttons for convenient settings adjustments, alarm pause and power on/off
- Up to 4 hours continuous use
- Adjustable Sweep speed
- Large animals and small animals airway adapters
- Selectable units mmHg, kPa, %
- Ultra lightweight
- Apnea detection and adjustable
- No annual routine calibration required
- Configurable alarms for Respiration rate alarm, EtCO2 high and low, clogged or missing adapter
- Support english, italian, french, hungarian, polish (other language upon request)
- Low battery indicator
- Bluetooth connectivity to export real time data



Check out our website at www.vet1.com.au or e-mail sales@vet1.com.au or call us on **1300 378 713** to discuss anything Veterinary Equipment related. We are Committed to Quality.



Vet1 VM10 Rapid Intervention Capnograph

Specifications

Mechanical and Environment

Dimensions	60 x 47 x 43 mm (2.36 x 1.85 x 1.69 inches)
Weight	~80 g (complete device with batteries), < 350grs with transport pouch.
Mechanical robustness	Meets the shock and vibration requirements for transport of SS-EN ISO 21647:2004 clause 21.102
Temperature and Humidity	Operating: -5 to 50°C, 10 to 90% RH, non-condensing Storage: -30 to 70°C, <90% RH, non-condensing
Atmospheric pressure	70 - 120 kPa

EtCO₂ Measurement

Transducer Type	Mainstream CO ₂ Monitor
Measuring method	Dual-wavelength, non-dispersive infrared spectroscopy
Initialization Time	Capnogram, displayed within few seconds at an ambient temperature of 25°C, and full specifications within 1-2 minutes especially if temperature below 15°C
CO ₂ Measurement Range	0 to 99 mmHg, 0 to 9.9%, 0 to 9.9 kPa
CO ₂ Accuracy	±(0.2% + 2% of reading)
Total system response time	< 1s
Calibration	No routine user calibration required. An airway adapter zero is required when changing to a new airway adapter
Recovery time after defibrillator test	Unaffected

Respiration Rate Measurement

Respiratory rate Range	0~150 Breaths Per Minute (BPM)
Respiration Rate Accuracy	±1BPM
Breath detect	Adaptive threshold, minimum 1 % CO ₂ change
Airway Adapters	< 5 cc dead space (large animals), < 1 cc dead space(small animals)

Animals Machin Interface

Display	Active OLED color graphic display, 65 000 colors, 128 x 128 pixels Multi-languages(English, French, Italian, Hungarian, polish and more to be supported)
Front Panel Buttons	4 buttons: power on/off, alarm pause, up/down selection
Alarms	No Adapter, Clogged Adapter, No Breath Detected, High AwRR, Low ETCO ₂ , High ETCO ₂ , Temperature out of range, Atmospheric pressure is over operating range, Low Battery



Vet1 VM10 Rapid Intervention Capnograph





Power Specifications

External Supply	None
Power Consumption	TBD
Batteries	Two AAA Cell batteries (2x1.5VDC): Alkaline IEC:LR03 or Energizer Ultimate Lithium L92 batteries. Use of other Lithium batteries may present a risk of fire or explosion
Battery life time	Duracell Plus Alkaline: ~4 hours Energizer Ultimate Lithium L92: ~7 hours

Regulatory Compliance

Certificate	CE
Standards compliance	IEC 60601-1:2012 IEC 60601-1-8:2012 IEC 60601-1-12:2014 ISO 80601-2-55:2018

Interfaaces

Part Number	Designation		Included with device
CapnoCOP	CapnoCOP module	1	
M401D-AL	Disposable Large animal airway adapter	1	
M401D-N	Small animal airway adapter	1	
RIC-P	Pouch	1	
RIC-L	Lanyard	1	
RIC-LI	Lithium battery	1	(contain)